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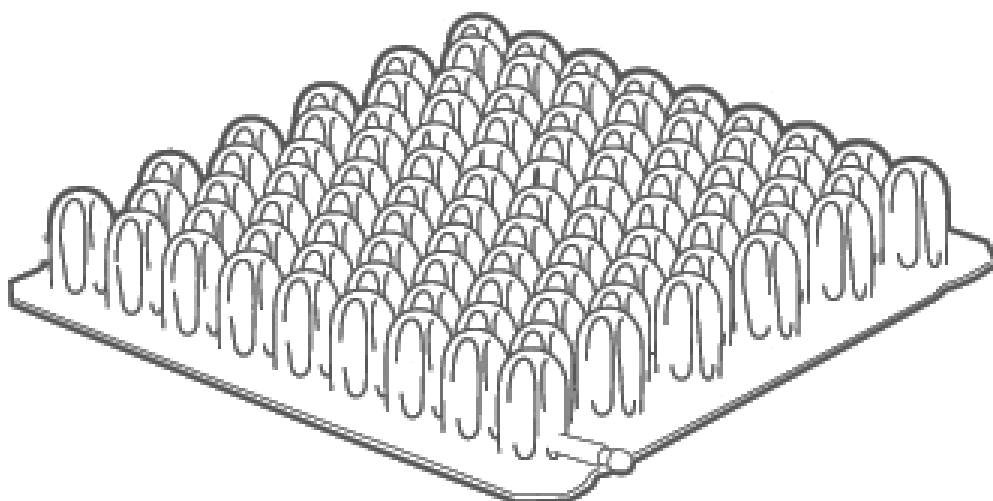
MANUAL DE USUARIO BASIC AIR


CBASIC4040

40X40X6,5CM


CBASIC4343

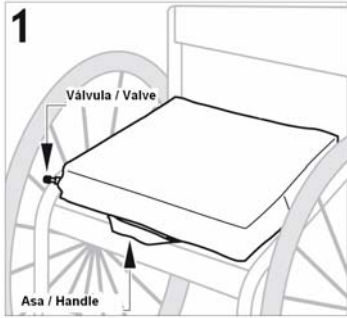43X43X6,5CM

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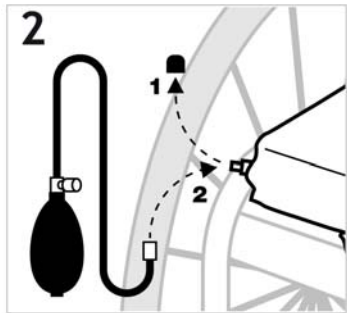
1 Place the cushion on the wheelchair with the valve positioned on the front right side (the handle of the cover must always be placed on the front).

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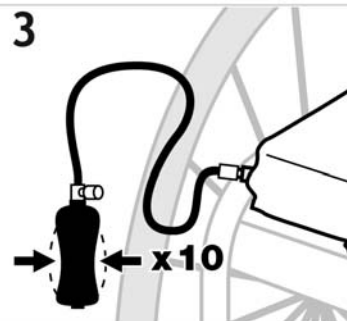
2. Remove the protection cup of the valve 1 and connect to the inflating device 2.

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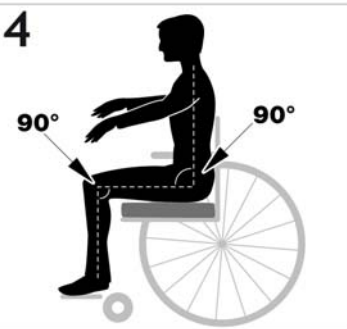3. Pre-inflate the cushion (press on the pump 10 times)

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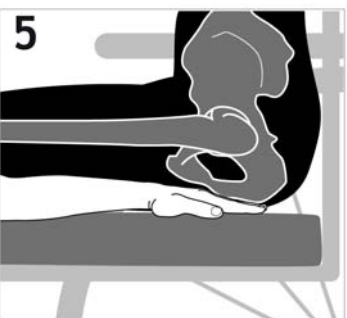
4. Place the patient on the cushion as follows:

Back placed against the back of the chair at a 90° angle.


Feet placed flat on the footrest which has been previously opened with the legs forming a straight angle.

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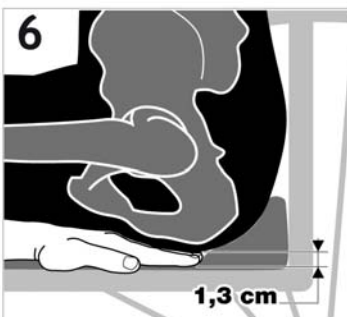5. Slide your hand between the patient's gluteus and the cushion; the latter must be placed under the ischium.

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6. Slowly deflate the cushion until the tips of your fingers can hardly move, so that there are approximately 1,3cm of air between the base of the cushion and the patient's buttocks.

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BASIC AIR

The first precaution to take against bedsores from being in a supine position involves changing the position of the body on the support every 3 hours on the average, while

English lifting the body is recommended for persons in a sitting position.

Relieve pressure points regularly whenever possible to avoid excessive prolonged compression of the blood vessels on bony areas and to facilitate circulation in the tissues.

Adjustable air cells cushion with one chamber. Removable protective cover.

WARNINGS AND PRECAUTIONS FOR USE

For reasons of hygiene, all medical devices used for the prevention of bedsores should be reserved for the exclusive use of one patient.

In the event of incontinence, the cover and eventually the cushion should be cleaned. It is advisable to purchase a second protective cover for use while the first is being washed, so as to avoid direct contact with the surface of the cushion.

Do not place any other materials or objects between the cushion and the patient's clothing, apart from the protective cover.

Seating cushions should not be used to support other parts of the patient's body (only the buttock zones). The ideal temperature for use is between -5°C and 45°C (between 23°F and 113°F).

STORAGE

- Place the cushion in a horizontal position.
- Do not expose it to sources of high heat.
- If a cushion is not used for an extended period of time, place it in a dry place with moderate temperature, away from light and heat.

HOW TO USE THE CUSHION

For a good seating position in the chair ensure that:

- The size of the cushion suits the chair. It is preferable not to use the cushion on a steeply inclined surface. Ensure that the correct positioning of front, back, top and bottom are respected.

MAINTENANCE AND CLEANING

DO NOT IRON, DO NOT USE CORROSIVE PRODUCTS OR ABRASSIVE MATERIALS

Cushion:      Wash in soapy water, rinse in clear water. Dry in open air.

Covers:      Wash at 40°C. Do not use bleach. Do not iron. Dry clean with P-type solvents. Trumble dry at moderate temperature.

DISINFECTION AND DECONTAMINATION

Disinfect using cold pulverization products (quaternary ammonium, glutaraldehyde, etc.).

INSTRUCTIONS FOR THE USE OF THE BASIC AIR REPAIR KIT

- Clean the surface to be repaired
- Deflate the cushion and mend the puncture with the glue
- Let dry for 1 hour

IMPORTANT: Do not glue any patches to the cushion.

RESPECT FOR THE ENVIRONMENT

Do not burn or dispose of in the environment. 0% CFCs.

Repair kit: Do not burn or dispose of in the environment. Highly inflammable. Keep the recipient closed in a well ventilated area. Keep away from any flame. Keep Away From Children. Do not smoke nearby. Do not breathe in the fumes.

SAFETY

The basic air cushion complies with applicable anti - fire regulations.

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c/Llobregat, 12 Polígono Industrial URVASA
08130 Santa Perpètua (Barcelona) ESPAÑA
tel. 935 74 74 74 fax. 935 74 74 75

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